

**INSTALLATION QUALITY CHECK SHEET  
FOR  
DIAGNOSTIC ULTRASOUND SYSTEM**

***Power**vision  
6000*

**MODEL SSA-370A  
(2J730-019E\*B)**

**TOSHIBA CORPORATION**

© TOSHIBA CORPORATION 1998

ALL RIGHTS RESERVED

**IMPORTANT!**

1. No part of this manual may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this manual are subject to change without prior notice and without our legal obligation.



Installation Quality Check Sheet

Model name : SSA-370A Serial No. : \_\_\_\_\_

Customer name : \_\_\_\_\_

Subsidiary : (TAMS, TMSE, TAP, TCL, TMB)  
\* Indicate the responsible subsidiary

Distributor name : \_\_\_\_\_

Installation completion date (DD, MM, YY) : \_\_\_\_\_

Installed by : \_\_\_\_\_

Installation result : Pass/Fail

[Comments on the installation quality check]

---

---

---

---

\* Column "Result": Check (OK) or fill in with measurements.  
If some work has been done, indicate the type of work using the corresponding number shown below.

1. Replaced, 2. Repaired, 3. Adjusted, 4. Tightened, 5. Cleaned, 6. Lubricated,  
7. Other work done

No.	Item (Method)	Standards	Result [Work]
1	<u>Check the configuration.</u>	No missing parts, wrong parts, or damaged parts	[ ] [ ]
2	<u>Check the external appearance.</u>	No scratches or damage to the exterior	[ ] [ ]
3	<u>Check the installation conditions.</u> a) Line voltage ( $\pm 10\%$ )	Single-phase AC voltage is constantly output while the power supply is turned ON.	[ ] [ ]
	b) Grounding	Must be protectively grounded.	[ ] [ ]
4	<u>Check the functions.</u> a) Image display function	The image must be displayed correctly.	[ ] [ ]
	b) Display-scale change function	The display scale must be changed correctly.	[ ] [ ]
	c) Measurement function	The distance and area/circumference must be measured correctly.	[ ] [ ]
	d) Keyboard entry function	Characters must be entered from the keyboard correctly.	[ ] [ ]
	e) Initial setting function	When the NEW PATIENT switch is pressed, the system must be reset to the initial status.	[ ] [ ]
	f) Print function	The displayed image must be printed out on the printer.	[ ] [ ]
5	<u>Check the images.</u> a) Check the displayed images.	A normal and complete ultrasound image should be displayed.	[ ] [ ]
	b) Check the noise in the operating status.	No noise with a specific pattern should be seen.	[ ] [ ]
6	<u>Other</u> Instruct the user to avoid moving the system over a slope, a large height difference, or a carpet with extremely long fibers.	Instruction must be given to the user.	[ ] [ ]



## Verifying Explanations Regarding Safe Operation

Model name : SSA-370A Serial No. : \_\_\_\_\_

One of the responsibilities of the person in charge of product installation at your site is to verify that explanations are made regarding the safety and operating precautions and check items. Please also verify that an explanation regarding the following contents is presented.

Date of explanation (DD, MM, YY)	Contents	Person who provided explanation	Person who received explanation
	The following sections of the operation manual for the main unit (2B730-556E or 557E) 1. Safety precautions 2. Important information	_____ _____	_____ _____
Confirmation signature (Administrator in hospital)			

# **TOSHIBA**

**TOSHIBA CORPORATION**

---

1385,SHIMOISHIGAMI,OTAWARA-SHI,TOCHIGI-KEN 324-8550,JAPAN